



Zenith Flex[®]

AAA ENDOVASCULAR GRAFT

DEVICE PLANNING AND SIZING WORKSHEET


Anatomical Measurements

Main Body Introduction Site

- ☐ Right iliac External iliac (EI) measurement _____ mm
- ☐ Left iliac External iliac (EI) measurement _____ mm

Table Position

- _____ Lowest renal artery
- _____ Aortic bifurcation
- 15 mm below lowest renal artery, check for 10% increase in diameter. _____ Origin cl internal iliac
- _____ Origin il internal iliac

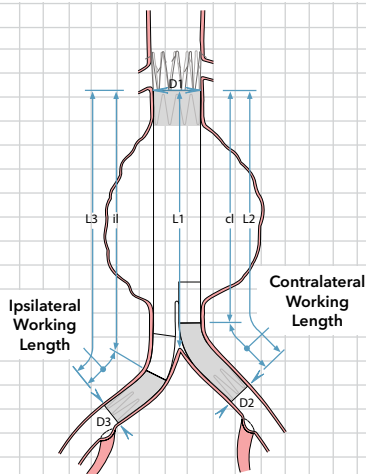

Diameters: D1 D2 D3

Lengths: L1 L2 L3

When using CT for length, approximate lateral deviation/tortuosity and add to difference in table position.

If choice of graft diameter or graft length is affected by other considerations, adjust accordingly.

Main Body

Main Body Diameters

D1 mm	Graft Diameter mm
18-19	22
20-21	24
22	26
23-24	28
25-26	30
27-28	32
29-32	36

Graft Diameters 22-32

L1 mm	cl Length ¹ mm	il Length ² mm
88-103	82	112
104-118	96	126
119-133	111	141
134-148	125	155
149-163	140	170

Graft Diameter 36

L1 mm	cl Length ¹ mm	il Length ² mm
101-120	95	125
121-139	113	143
140-158	131	161
159-177	149	179

¹Main body length on contralateral side

²Main body length on ipsilateral side

Main Body Order Number = TFFB- - - ZT

Graft Diameter cl Length/
Graft Length

Date: _____ Patient ID: _____

Hospital: _____

Physician Name: _____

Physician Phone #: _____

Physician E-mail: _____

Physician Signature: _____

Contralateral Spiral-Z[™] AAA Iliac Leg

$$\text{L2} \text{ minus } \text{cl Length} = \text{Contralateral Working Length}$$

Contralateral Leg (ZSLE) Diameters

D2 mm	Graft Diameter mm
≤ 8	9
9	11
10-12	13
13-15	16
16-18	20
19-20	24

Contralateral Leg (ZSLE) Lengths

Contralateral Working Length mm	Graft Length mm	Recommended Overlap mm
27-43	39 ³	22-30
44-60	56	22-30
61-77	74	22-30
78-94	90	22-30
95-111	107 ⁴	22-30
112-122	122 ⁴	22-30

³Assure adequate distal fixation length.

⁴Graft lengths of 107 and 122 mm are available in 9-13 mm diameters only.

Contralateral Leg Order Number = ZSLE- - - ZT

Graft Diameter Graft Length

Ipsilateral Spiral-Z AAA Iliac Leg

$$\text{L3} \text{ minus } \text{il Length} = \text{Ipsilateral Working Length}$$

Ipsilateral Leg (ZSLE) Diameters

D3 mm	Graft Diameter mm
≤ 8	9
9	11
10-12	13
13-15	16
16-18	20
19-20	24

Ipsilateral Leg (ZSLE) Lengths

Ipsilateral Working Length mm	Graft Length mm	Recommended Overlap mm
20-35	39	22-30
graft diameter 20-24		
20-35	56	22-55
graft diameter 9-16		
36-42	56	22-30
43-59	74	22-55
60-76	90	22-55
77-93	107 ⁴	22-55
94-122	122 ⁴	22-55

⁴Graft lengths of 107 and 122 mm are available in 9-13 mm diameters only.

Ipsilateral Leg Order Number = ZSLE- - - ZT

Graft Diameter Graft Length

EMEA: EDI - www.cookmedical.com/edi.do
Distributors: +353 61239240, ssc.distributors@cookmedical.com
Austria: +43 179567121, oe.orders@cookmedical.com
Belgium: +32 27001633, be.orders@cookmedical.com
Denmark: +45 38487607, da.orders@cookmedical.com
France: +33 171230269, fr.orders@cookmedical.com
Germany: +49 6950072804, de.orders@cookmedical.com
Hungary: +36 17779199, hu.orders@cookmedical.com
Ireland: +353 61239252, ie.orders@cookmedical.com
Italy: +39 0269682853, it.orders@cookmedical.com
Netherlands: +31 202013367, nl.orders@cookmedical.com
Norway: +47 23162968, no.orders@cookmedical.com
Poland: +48 223060159, pl.orders@cookmedical.com
Spain: +34 912702691, es.orders@cookmedical.com
Sweden: +46 858796468, se.orders@cookmedical.com
Switzerland - French: +41 448009609, fr.orders@cookmedical.com
Switzerland - Italian: +41 448009609, it.orders@cookmedical.com
Switzerland - German: +41 448009609, de.orders@cookmedical.com
United Kingdom: +44 2073654183, uk.orders@cookmedical.com

Americas: EDI - www.cookmedical.com/edi.do
Phone: +1 812.339.2235, 800.457.4500, Fax: 800.554.8335
E-mail: orders@cookmedical.com

Australia:
Phone: +61 738411188, 1800777222, Fax: +61 738411288, 1800077283
E-mail: cau.custserv@cookmedical.com

© COOK 2010 ALBPH-FXSZPSW-EN-201012

WOMEN'S HEALTH

